



Now available:
bubble gum



VOCO Profluorid® Varnish

FLUORIDE-CONTAINING DENTAL DESENSITISING
VARNISH



VOCO Profluorid® Varnish

HIGHLY EFFECTIVE



Sensitive teeth are a widespread problem for a large number of people. Usually, the condition causes only transient pain which patients do not always report to their dentists. Hypersensitivity is mainly caused by exposed dentinal tubules. Therefore, the aim of treatment is to seal these tubules. Profluorid Varnish is a fluoride-containing varnish to effectively seal dentinal tubules exposed on the necks of teeth or after cavity preparation.

Components and effect of VOCO Profluorid® Varnish

Profluorid Varnish is a colophony-based varnish containing 5 % sodium fluoride (22,600 ppm fluoride). The fluoride ion, together with the calcium ions accumulated in the tubules, causes a precipitation of calcium fluoride, which effectively seals the tubules. According to a user survey, over 85 % of patients were completely free of pain after only one treatment with Profluorid Varnish.

As well as rapidly sealing the dentinal tubules, Profluorid Varnish also causes calcium fluoride deposits to form on the tooth surface and contributes to the formation of fluorapatite in the long-term. The fluorapatite protects the tooth from acid attacks.

Xylitol, which has a proven cariostatic effect in addition to its taste-enhancing properties, is also added to the varnish.

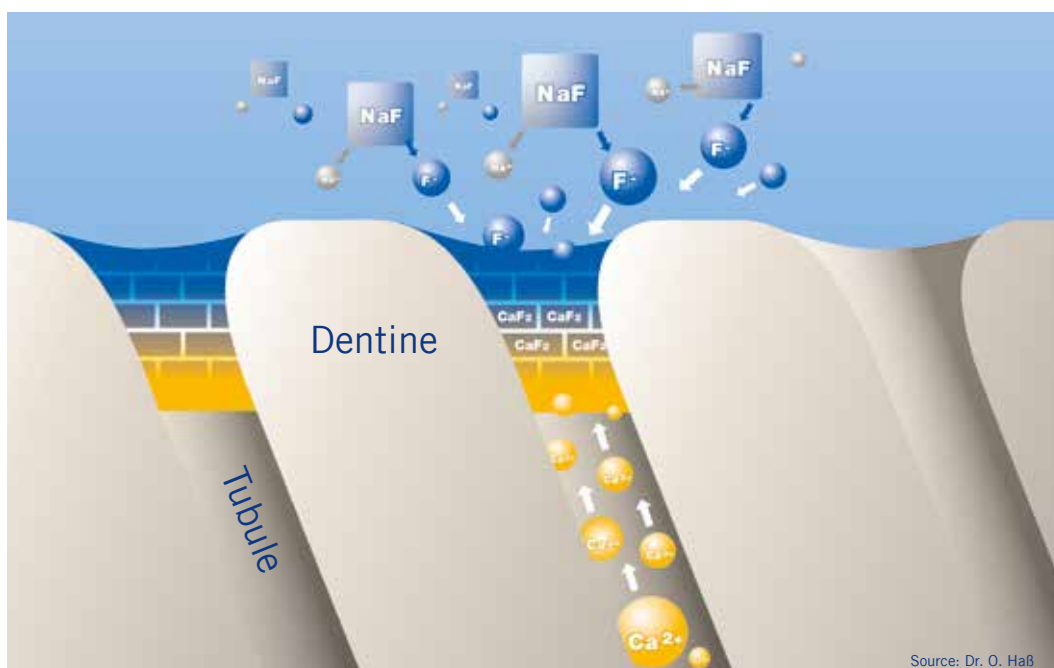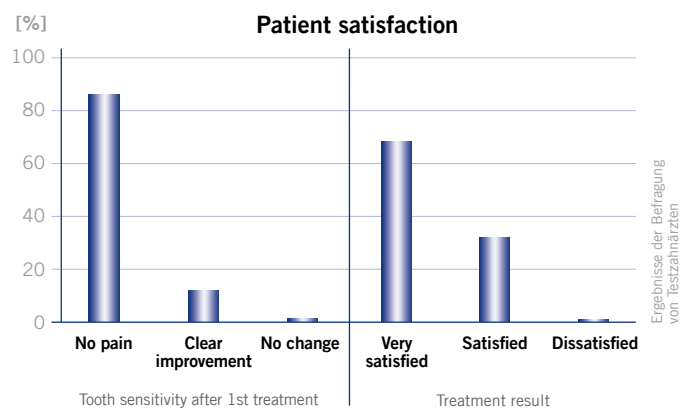

VOCO Profluorid® Varnish


SIMPLE AND AESTHETIC

Easy to apply

Thanks to its colophony matrix, Profluorid Varnish adheres well even to wet surfaces. It is not necessary to dry the tooth surfaces. This improves cooperation of patients with low levels of compliance (e.g. children).

Aesthetics and taste

Profluorid Varnish is a tooth-shaded varnish which does not impair aesthetics. Profluorid Varnish is available in five pleasant flavours: bubble gum and cherry – aimed especially at children and younger patients, mint – a classic, particularly for adults, caramel as a creamy flavor and melon – to provide a slightly more exotic option. These five refined flavours will enable you to provide an even better service by tailoring it to the individual patient.

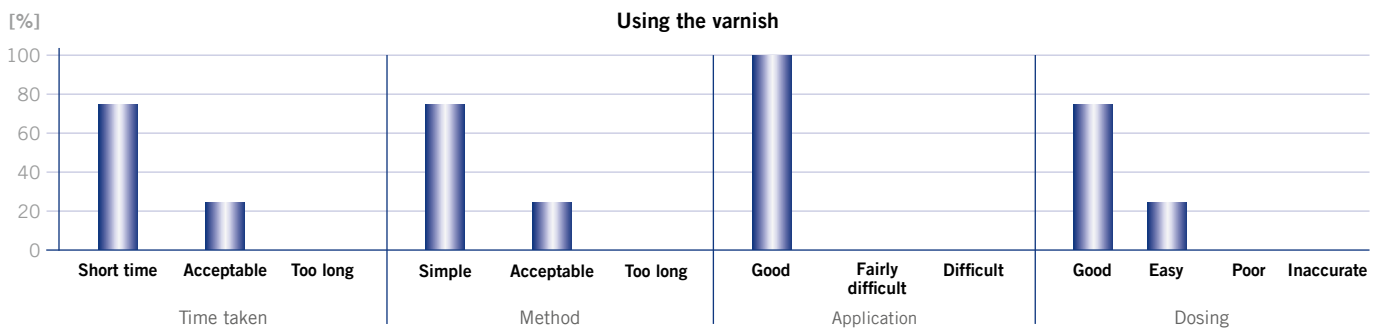

Survey of testing dentists

Opinion of dentists

Dentists using the product all confirmed its good handling and aesthetic properties. A vast majority also reported positive differences between Profluorid Varnish and competing products. Profluorid Varnish was equally well accepted by the patients.

After the treatment, nearly all of the patients reported that they were either symptom-free or had a significantly reduced level of sensitivity.

Profluorid Varnish: much more than just good taste!

Simple and aesthetic desensitising with VOCO Profluorid® Varnish



The *SingleDose* advantages
Profluorid Varnish *SingleDose* comes in a blister pack containing a brush and the varnish. This enables the product to be applied under perfectly hygienic conditions.

Perfect application with VOCO Profluorid® Varnish



The cartridge advantages
The tried and tested cartridge enables you to apply the material with pinpoint accuracy, and to fluoride a complete dental arch in one go.

VOCO Profluorid® Varnish

YOUR CHOICE

Indications

Treatment of hypersensitive teeth

Sealing the dentinal tubules, e.g. for cavity preparations or on sensitive root surfaces

Advantages

- Easy to apply on wet tooth surfaces
- Esthetic, tooth-shaded varnish
- Fast desensitisation and fluoride release (5 % NaF \triangleq 22,600 ppm fluoride)
- In five pleasant flavours available: melon, mint, cherry, caramel and bubble gum
- Economical application in thin layers, high yield
- Ideal for treating cervical areas after professional cleaning and calculus removal

Advantages of the *SingleDose*

- Easy, quick and hygienic
- No additional devices required
- No spilling in any position

Advantages of the cartridge

- Multiple use with disposable cannulae
- Perfectly precise application
- Extremely easy application in proximal areas
- Complete fluoride application in only a single step



Presentation

REF 2971 Tube 4 x 10 ml mixed

Flavour	SingleDose 48 x 0.40 ml	SingleDose 50 x 0.25 ml	SingleDose 50 x 0.40 ml	SingleDose 200 x 0.40 ml	Tube 10 ml	Tube 50 ml	Tube 4 x 10 ml	Cartridge 5 x 1.70 ml
Melon	–	REF 1268	REF 1269	REF 1271	REF 1267	REF 1272	–	REF 2235
Caramel	–	–	REF 2226	REF 2229	REF 2232	–	–	–
Cherry	–	–	REF 2227	REF 2230	REF 2233	–	–	–
Mint	–	–	REF 2228	REF 2231	REF 2234	–	–	–
Bubble gum	–	–	REF 2238	REF 2239	–	–	–	–
Mixed	REF 2225*	–	–	–	–	–	REF 2971	–
REF 2167	Application cannulae type 43, 100 pcs.					*(12 each of melon, caramel, cherry, mint)		

VOCO GmbH
Anton-Flettner-Straße 1-3
27472 Cuxhaven
Germany

Freecall: 00 800 44 444 555
Fax: +49 (0) 4721-719-140

info@voco.com
www.voco.dental

Available from: